

## Operating instructions for handpiece


### Inserting and changing burs (also see drawing below)

Ensure that the bur motor is turned off before inserting or changing the bur. **The motor must only be turned on when the bur has been secured.** If this is not done, an automatic fuse will respond and the handpiece will not operate.

By turning the milled nut to the right, the chuck opens allowing the bur to be inserted. (Ensure that the shaft of the bur is clean and dry before inserting.) By turning the milled nut back to the left, the chuck closes and the bur is secured.

Before inserting the bur slide one of the rubber dust caps over the shaft of the bur. The bur is then secured as described above. The distance between the rubber dust cap and the hand set is about the thickness of a nail. The bur then rotates freely without the dust cap touching the handset.

**This is to prevent entering dust and moisture into the ball bearings**

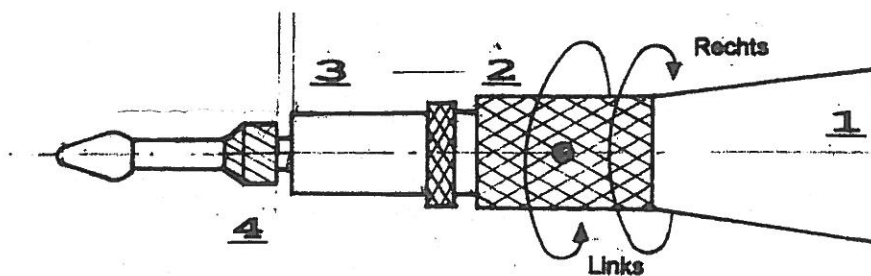

Additional important points

- \* Do not drop the handpiece
- \* Burs with a diameter of more than 8 mm must not be used.
- \* Do not use oil or any other lubricant in the handpiece and unit.
- \* Do not use compressed air to clean the handpiece as this can blow dirt into the handpiece
- \* Orthofex recommends having the handpiece and unit serviced once a year.
- \* Always use dust caps to prevent entering moisture into the ball bearings

The warranty expires if you do not follow these rules